

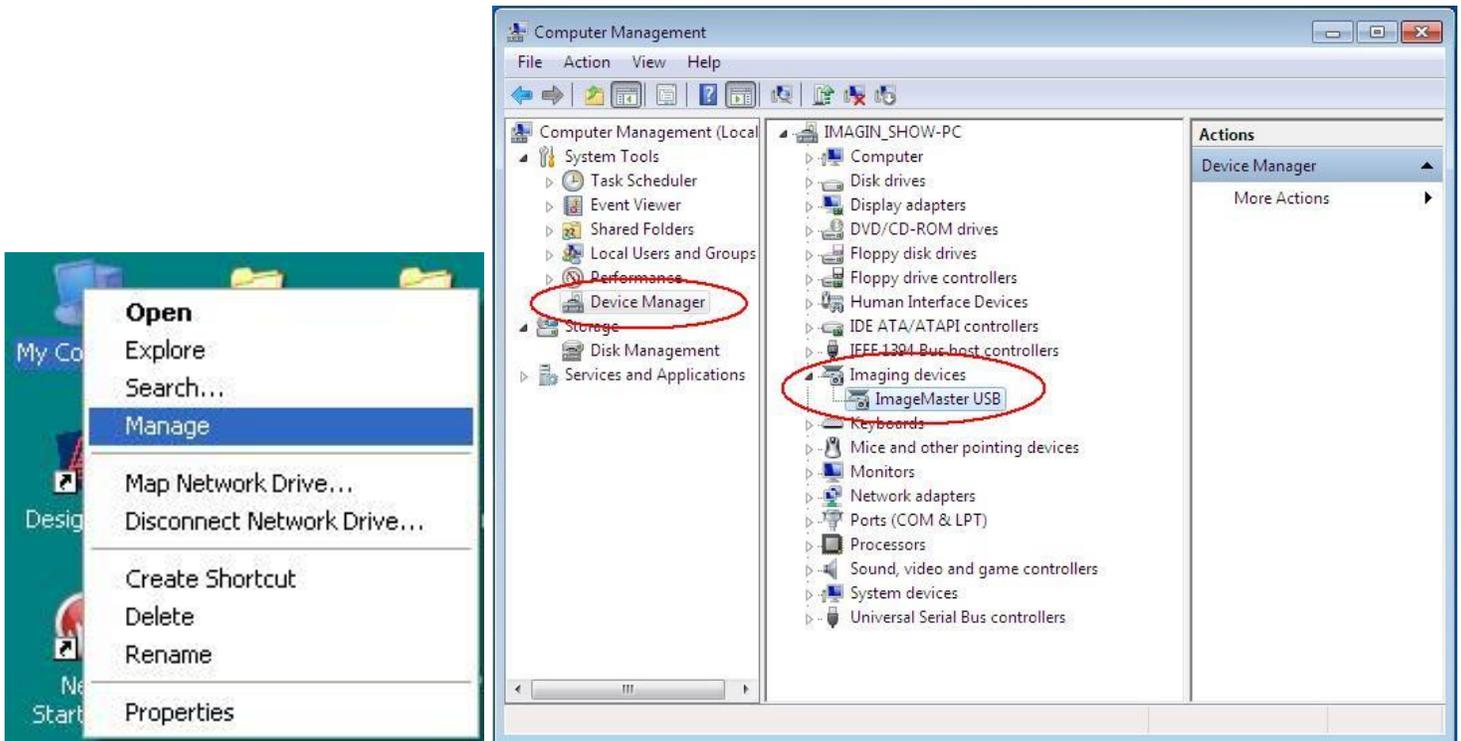


## ImageMaster USB Camera Settings in Dentrux Image:


1. Connect camera to the computer's USB 2.0 Port, wait for 30 seconds until a message at the bottom of the screen pops up that says "Found New Hardware and it is Ready to Use"
2. Insert the ImageMaster Resources CD into the Computer's CD-ROM Drive, then RIGHT click the mouse on the My Computer icon on the Desktop.
3. Select *Manage* → *Device Manager* → *Imaging Devices*

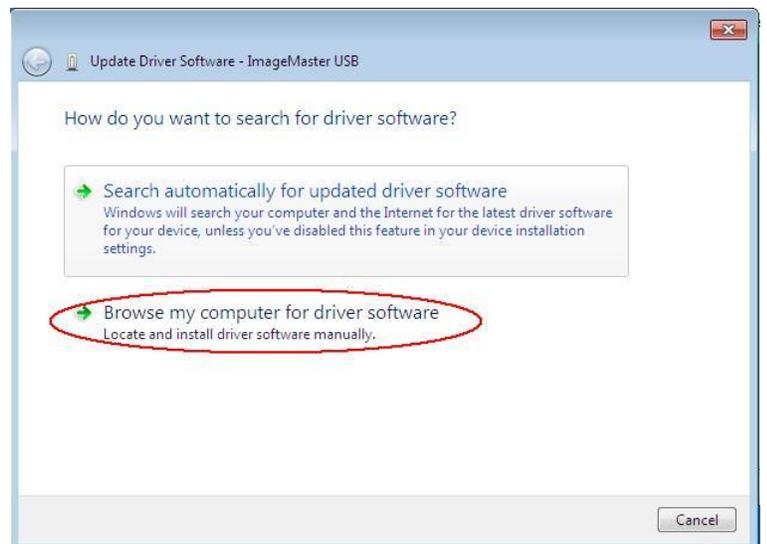

→ RIGHT click on *ImageMaster USB* → *Update Driver Software* → *Browse my computer for driver software*

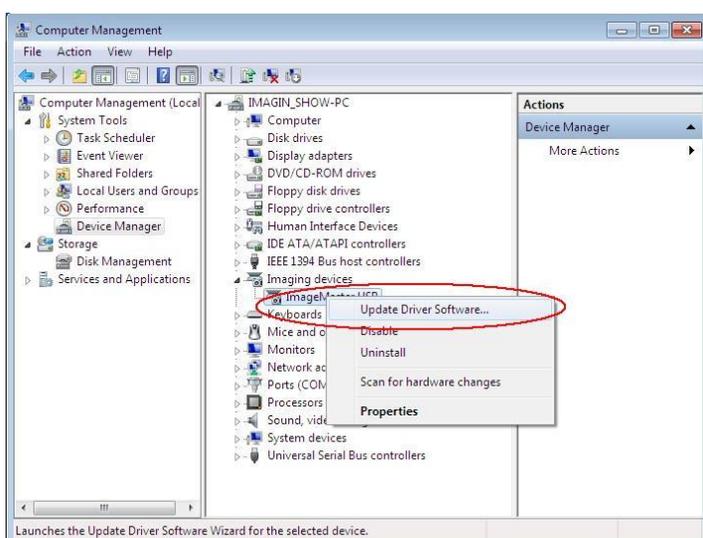

→ Browse → select *CD-rom drive* → *Dentrix Image* → *32 bit Driver* → then click on "OK"

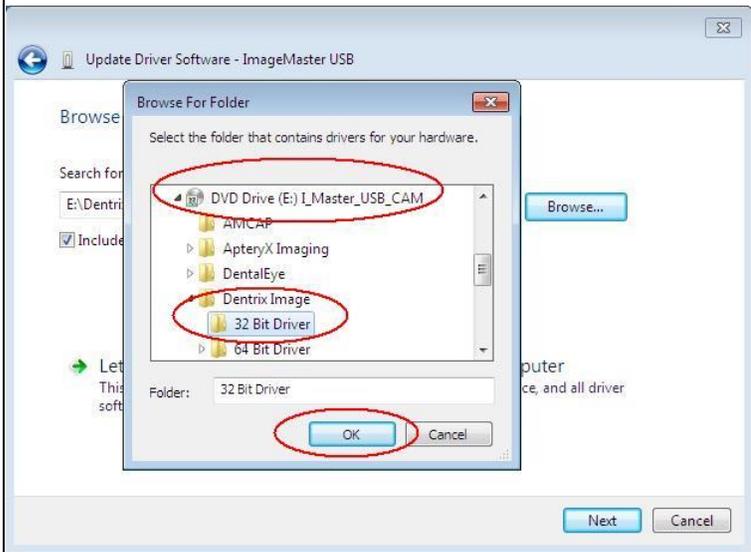

→ then click on *Next* → *Install this driver software anyway.*

