

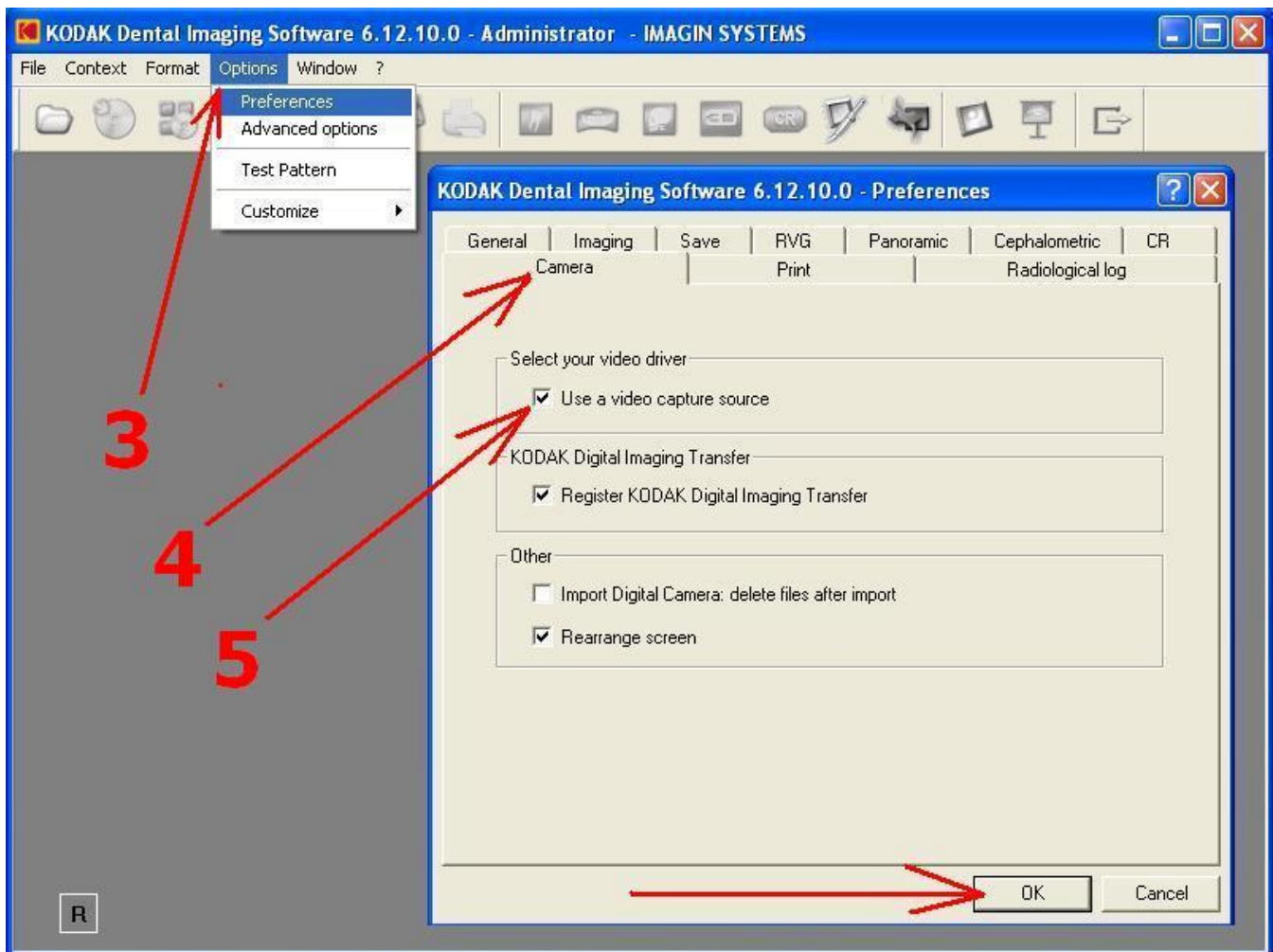
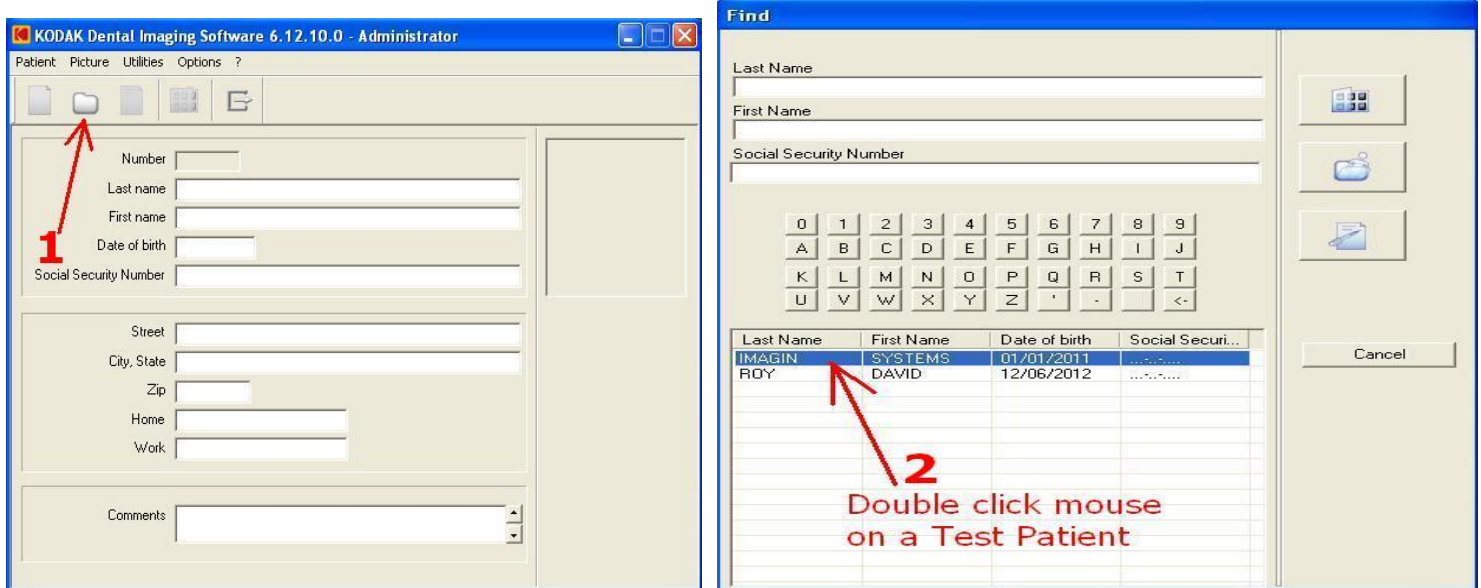


ImageMaster USB Camera Settings in Kodak Imaging:

1. Connect camera to computer's USB port, then open Kodak Imaging Software, select a patient, click on *Options* → *Preferences* → *Camera* → check the box "use a video capture source" → then click OK.

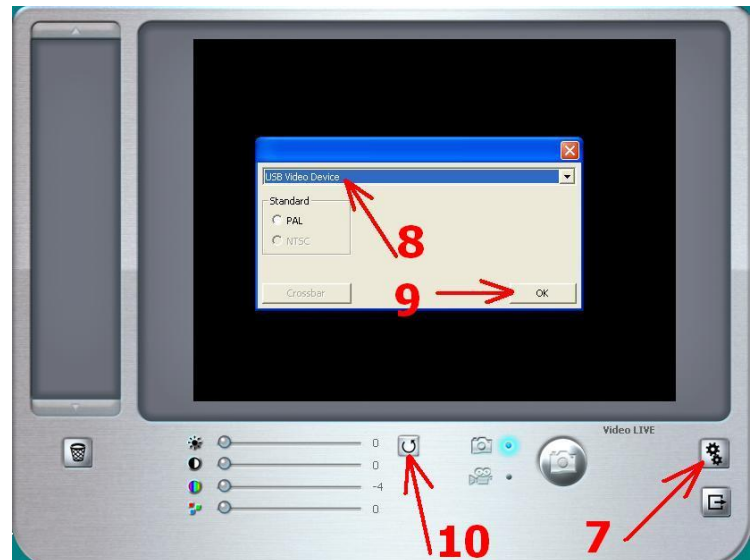

2. Then select "Camera" in the drop down menu bar on step 8:
-For Windows XP select *USB Video Device*
-For Windows 7 select *ImageMasterUSB*

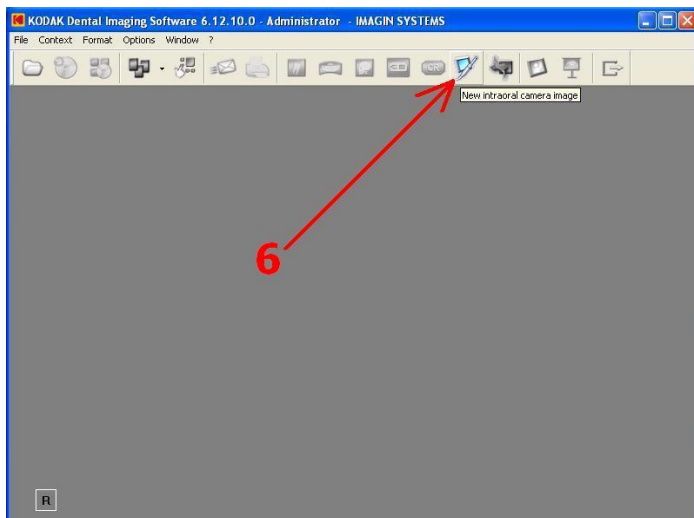

The camera is now ready for capturing images. Press once on the camera capture button to freeze an image on the screen, press the button again to save.

