

## ImageMaster USB Camera Settings in Schick Imaging:



1. First install the Video Filter from the CD that comes with the camera.

Insert the CD into the computer CD-rom drive, open the Video Filter folder, double click on the Video Filter setup file → *click on Run* → *I Accept* → *uncheck 2 boxes*, & *click on Decline* → *Uncheck box*, & *click on Decline* → *Finish*.





2. Connect the camera to the computer's USB 2.0 port.

Double click on the ManyCam icon on desktop, click on *Sources* → *Cameras* → *ImageMaster USB* or (USB Video Device if Windows XP) → 640 x 480 30fps

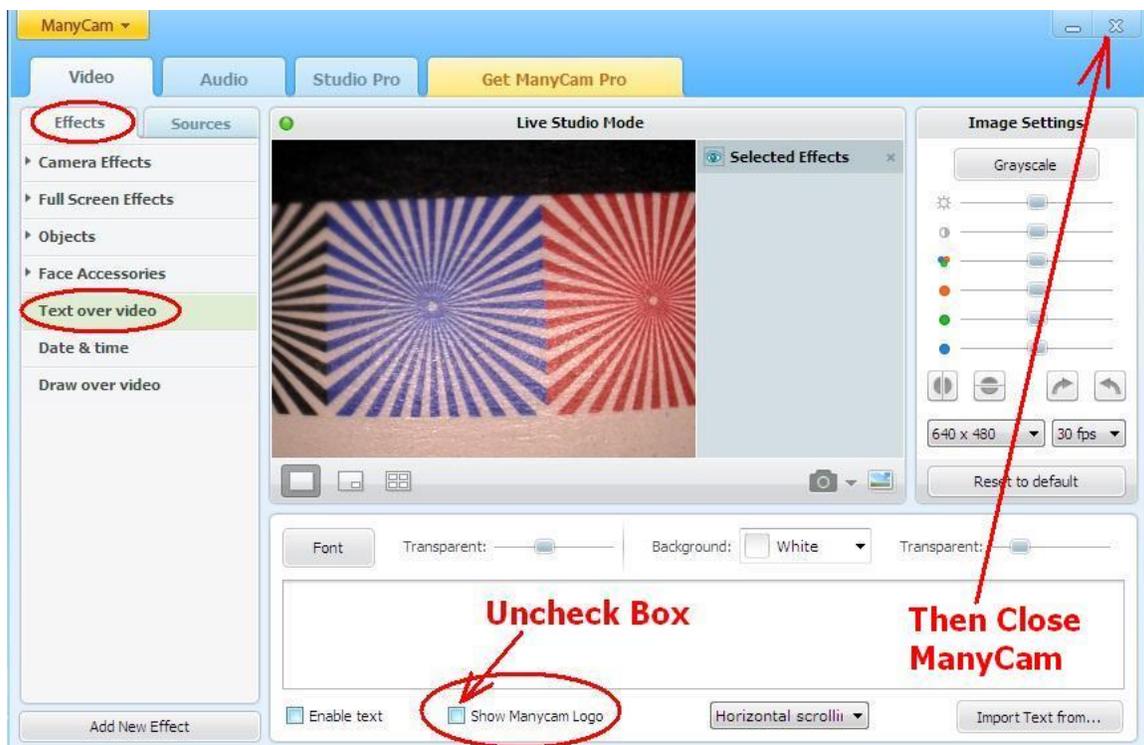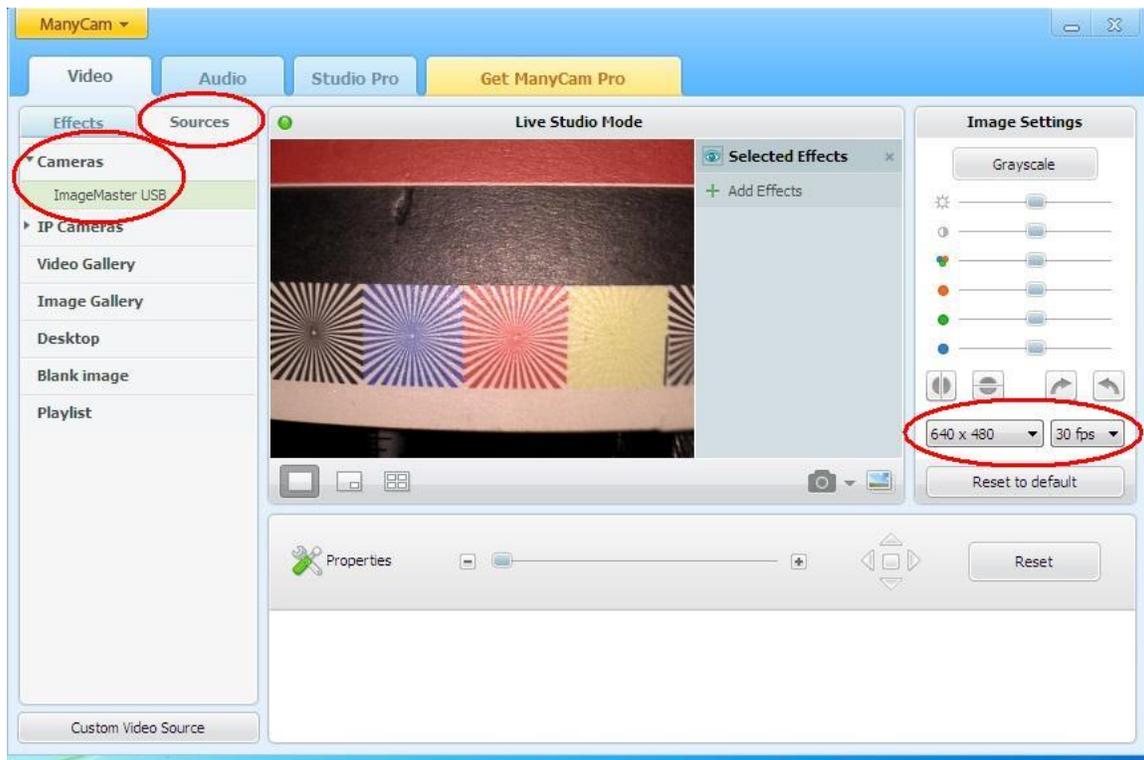

### 3. Open and Log onto Schick Imaging (CDR).

Click on Open Patient → select a Test Patient → then click on Xray/Image  
→ Acquisition Mode → check Video. Then click on Xray/Image again → Capture Video

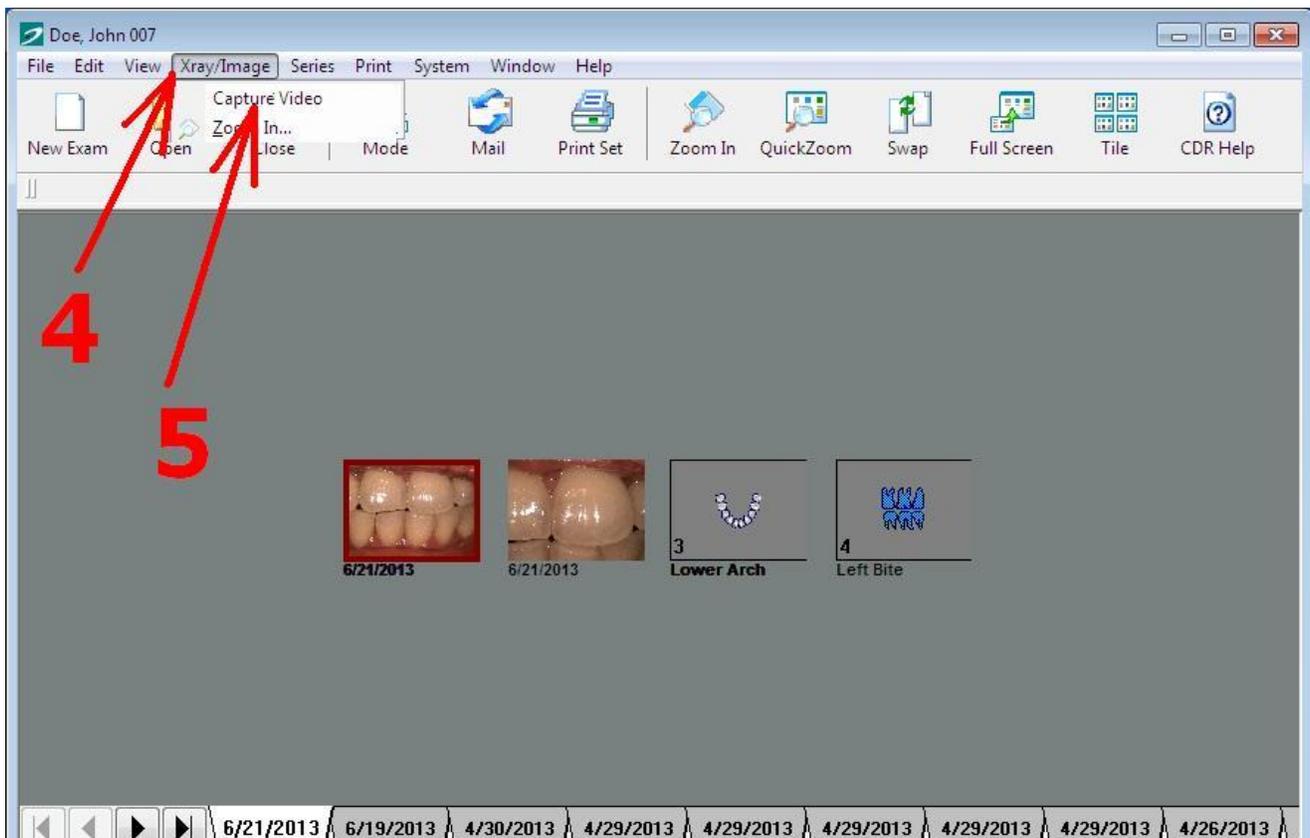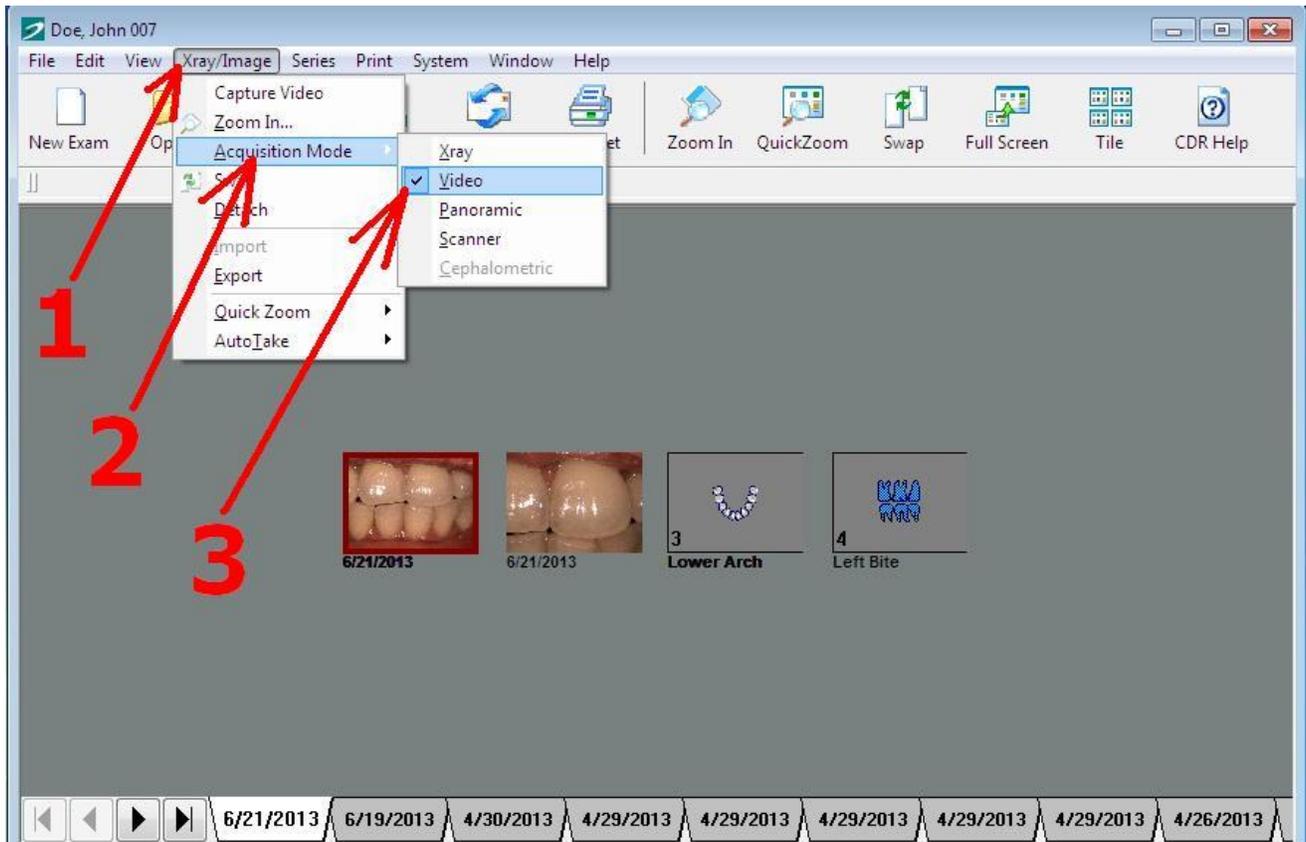

4. Then click on Setup → Change Driver → and select ManyCam Virtual Webcam → OK.

