
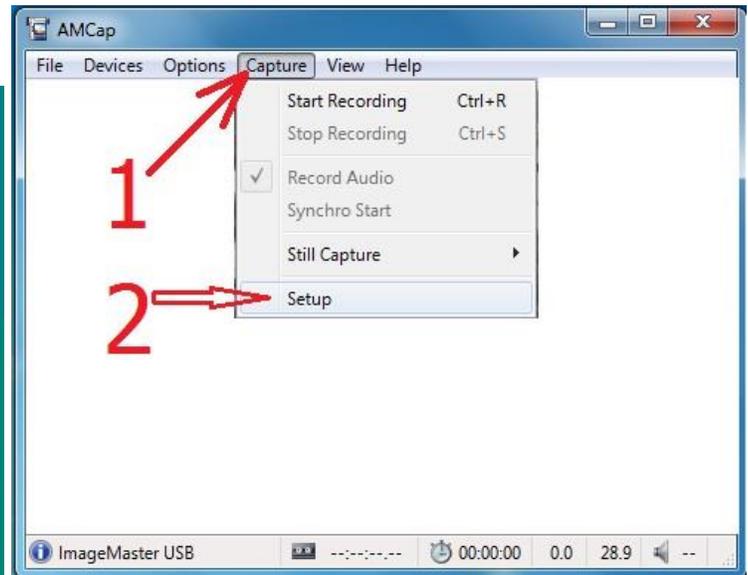




## ImageMaster USB Camera Settings in AMCap:

1. Create an AmCap folder for saving Images:

RIGHT click the mouse anywhere on the desktop → *New* → *Folder and type "Camera Images"*  
Then open AMCap and go to *Capture* → *Setup*

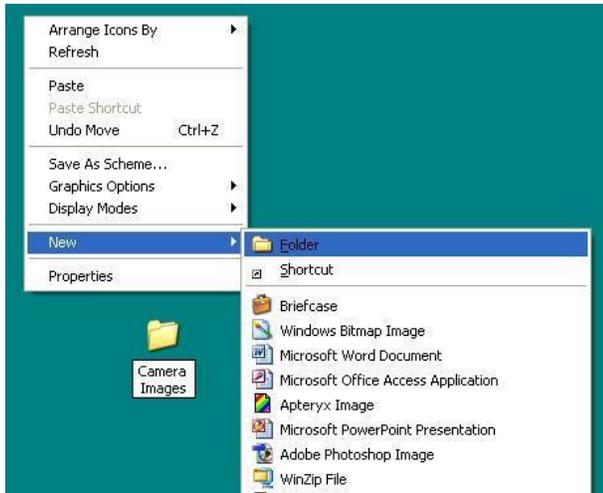

Click on *General* → **check** the two boxes shown in step number 3 →  
*Still Capture* – make sure all three boxes are checked as shown in step number 5 below → OK

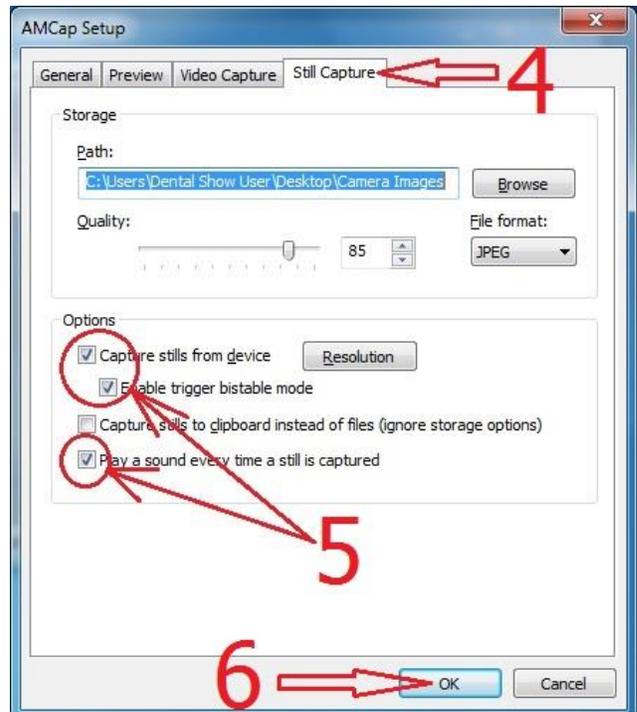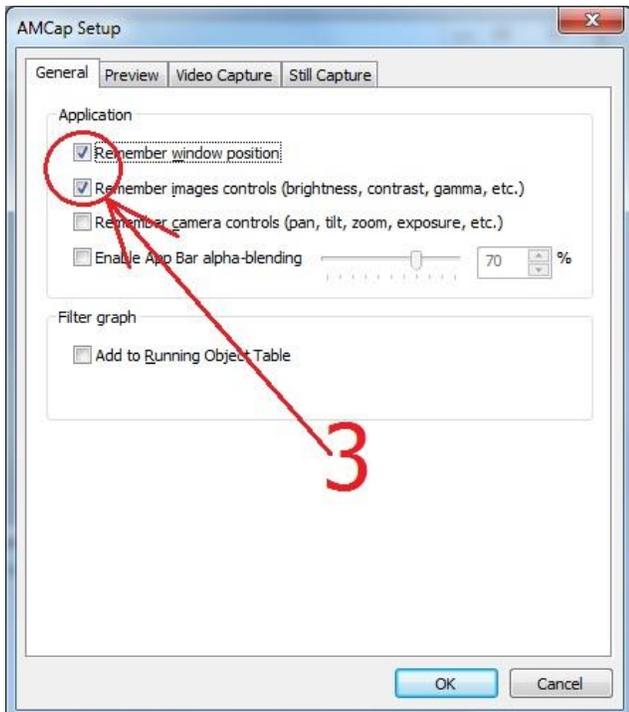

2. Follow the steps below for adjusting the color in AmCap:

*Options* → Video Device → Video Capture Filter as shown in step one, next *Properties* → video Proc AMP → change the contrast to 5 and the white balance to 3800, then select OK.

