

ImageMaster Camera Capture Button Up-Date Settings for ApteryX Imaging software:

1. Up-dating older versions of Ateryx: first close all CPUs running Apteryx and exit Name Grabbers:



- 2. Copy these 2 files "ATVIDEO.DLL" and "DIRECTVIDEO.IDDLL" from the CD on to your computer, (the default location would be: C:\Program Files\Apteryx\Apteryx Imaging\)
- 3. Close all folders, and open XrayVision \rightarrow DirectVideo, and select Video Options \rightarrow Still Pin Capture.



4. Config Capture Button: go to Video Options → Advance Options → Capture Button Assignment → Check box 3



5. Adjusting Color:

Go to "Video Options", then select "Video Capture"

Change the contrast to **5**, sharpness to **2**, and white balance to **3798**, then click on OK.

🗙 XrayVisi	on								
File Patient	Database	DICOM	TransIT	ArchiveIT	Tools Wind	tow Help			
Open	Save	ND Open Patient	O _ੈ Add Patient	Print	Pant Mulitple	Undo	Ca Redo	Capture Single	Capture Layout
1	Direc	tVideo							
	Video Dev	ne Vic	leo Options Advanced Still Pin An Video Capt Video Capt	Images Options d Other Vide ture ture Pin	Capture But	tons About		1	
							2		

